Trak Back II Disposable Pullback Device

Figure 1 : Illustration of Trak Back II Device

CAUTION: U.S. Federal Law restricts this device to sale, distribution and use by, or on the order of a physician.

WARNING:

- The VOLCANO Trak Back II device is for single use only. Do not re-sterilize and/or re-use the device.

CAUTIONS:

- Caution statements identify conditions or practices that could result in erroneous measurement during the use of this device.
- This product is in compliance with the immunity requirements of EN 60601-1-2: 1993, and is in accordance with the EN 55011:1998 Class A radiated emissions limits.
- The Trak Back II device is to be used only with the Avamar® or Eagle Eye® series of catheters. Use with Visions® family of catheters is contra-indicated.
- This product has no specific disposal requirements other than for biohazard hospital wastes.

CONTENTS:

1 Trak Back II device

Carefully read this package insert prior to use. Observe all warnings and precautions noted throughout these instructions. Failure to do so may result in complications.

DESCRIPTION:

The Volcano Corporation (VOLCANO) Trak Back II is a disposable pullback device designed for use with the VOLCANO In-Vision Ultrasound Imaging System.

Trak Back II is used to move the catheter steadily and precisely, allowing uniform collection of imaging data.

PREPARATION FOR USE:

Prior to opening the Trak Back II sterile packaging, prepare the patient for intravascular ultrasound imaging. Using standard intraoperative or percutaneous techniques, introduce the catheter into the patient and advance it so that it is distal to the lesion or region to be examined.

Prior to performing a pullback, make certain there is no resistance to catheter motion by moving the catheter manually over the region to be imaged.

INSTRUCTIONS FOR USE:

Remove the Trak Back II from its sterile packaging and place it near the catheter entry point of the patient.

CAUTION: Remove the plastic protective piece from between the roller and drive wheel before using the Trak Back II device.

CAUTION: Do not over tighten the rotating hemostatic valve. Doing so may restrict motion and affect smoothness of movement.

Push the lift tab of the Trak Back II toward the top of the device label and open the lid. Remove the protective plastic piece from the roller and drive wheel. Assure that Trak Back II is in working condition by turning motor on for approximately one minute. Place the Trak Back II in a position at approximately the middle of the thigh, at least 5cm from rotating hemostatic valve. Loosen the rotating hemostatic valve enough to assure smooth movement of catheter during pullback. Place catheter into slot in Trak Back II, over roller. Position guidewire outside of the Trak Back II so that it does not get caught in the device when the lid is closed. Remove the slack of catheter in vessel by pulling catheter until catheter tip just begins to move. Carefully close the lid of Trak Back II, assuring that catheter remains in slot and over roller, and that guidewire remains outside of the device.

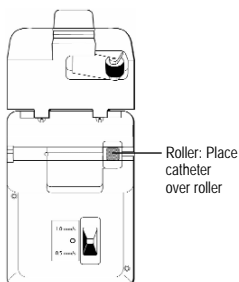
Figure 2 : Trak Back II with lid open

CAUTION: Once the Trak Back II lid has been closed, do not move the device away from the patient. This can cause the loss of guiding catheter position, if the rotating hemostatic valve is pulled.

Prior to the start of pullback, hold guidewire close to rotating hemostatic adaptor and hold Trak Back II in place. Start pullback at desired speed. Assure that catheter tip is moving prior to recording intravascular ultrasound data. Continue to hold guidewire and Trak Back II during the entire pullback.

CAUTION: Do not run the motor of the Trak Back II for more than 20 minutes. Doing so may run down the battery, causing the device to automatically shut down.

STORAGE:

Store in a cool, dry place.

NOTICE: Manufacturer's Specifications and Policies Subject to Change. VOLCANO reserves the right to make changes in the products described in this manual in order to improve design or performance. Reproduction or distribution of any portion of this manual without the prior written consent of VOLCANO is prohibited.

LIMITED WARRANTY:

Subject to the conditions and limitations on liability stated herein, Volcano Corporation ("VOLCANO") warrants that the Disposable Pullback Device, Trak Back II (the "Device") as so delivered, shall materially conform to VOLCANO's then current specifications for the Device, for a period of one year from the date of delivery. ANY LIABILITY OF VOLCANO WITH RESPECT TO THE DEVICE OR THE PERFORMANCE THEREOF UNDER ANY WARRANTY, NEGLIGENCE, STRICT LIABILITY OR OTHER THEORY WILL BE LIMITED EXCLUSIVELY TO DEVICE REPAIR, REPLACEMENT OR, IF REPLACEMENT IS INADEQUATE AS A REMEDY OR, IN VOLCANO'S OPINION, IMPRACTICAL, TO REFUND OF THE PRICE PAID FOR THE DEVICE. EXCEPT FOR THE FOREGOING, THE DEVICE IS PROVIDED "AS IS" WITHOUT WARRANTY OF ANY KIND, EXPRESSED OR IMPLIED, INCLUDING WITHOUT LIMITATION, ANY WARRANTY OF FITNESS, MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE OR NON-INFRINGEMENT. FURTHER, VOLCANO DOES NOT WARRANT, GUARANTEE, OR MAKE ANY REPRESENTATIONS REGARDING THE USE, OR THE RESULTS OF THE USE, OF THE DEVICE OR WRITTEN MATERIALS IN TERMS OF CORRECTNESS, ACCURACY, RELIABILITY, OR OTHERWISE. Buyer understands that VOLCANO is not responsible for and will have no liability for any items or any services provided by any persons other than VOLCANO. VOLCANO shall have no liability for delays or failures beyond its reasonable control.

Additionally, this warranty does not apply if:

- The Device is operated in other than a manner prescribed by VOLCANO in the Operator's Manual, and/or supplements.
- The Device is operated in a manner that is not in conformance with purchase specifications and specifications contained in the Operator's Manual, and/or supplements.
- The Device is not maintained in accordance with procedures in the Operator's Manual, and/or supplements.
- The Device is repaired, altered, or modified by other than VOLCANO authorized personnel, or without VOLCANO authorization.

Contact VOLCANO for instructions and issuance of a Return Material Authorization if claims under this warranty become necessary and if the Device or components of the Device are to be returned. The Device or components will not be accepted for warranty purposes unless the return has been authorized by VOLCANO.

Device parts or components repaired or replaced under warranty bear the same warranty expiration date as the original equipment. Consumable parts (data disks, batteries, among others) are warranted only against defects in materials and workmanship. Device parts purchased outside the original warranty period are warranted for a period of 90 days, subject to all of the restrictions contained in this Limited Warranty.


Use of unauthorized replacement parts may void the warranty. In all cases, VOLCANO will be the sole judge as to what constitutes warrantable damage.

Trak Back is a registered trademark of Volcano Corporation in the United States and Europe. Avamar is a registered trademark of Volcano Corporation in the United States, Europe and Japan. Eagle Eye is a registered trademark of Volcano Corporation in the United States, Japan and Europe. Visions is a registered trademark of Volcano Corporation in the United States. Volcano and the Volcano logo are registered trademarks of Volcano Corporation.

This product, and the use thereof, may be covered by one or more of the following U.S. and international patent 6290675. Other U.S. and international patents pending.

This product is licensed to the customer for single use only. Any re-sterilization or subsequent re-use is an unlicensed use and therefore constitutes patent infringement.

Graphic Symbols for Medical Device Labeling

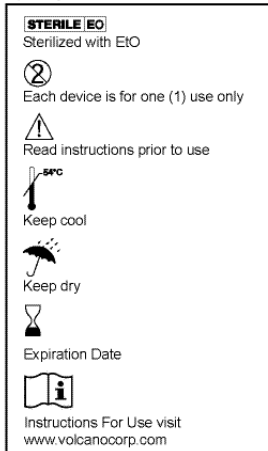Manufactured by:
Volcano Corporation
2870 Kilgore Road
Rancho Cordova, CA 95670
USA
(916) 638-8008
(800) 228-4728

Authorized European
Representative:
Volcano Europe SA/NV
Excelsiorlaan 41
B-1930 Zaventem Belgium
+32.2.679.1076
+32.2.679.1079 Fax

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